

Ostomy Care Product Catalogue

Ostomy care for skin, body and mind





What is a stoma?

A stoma is an artificial opening in the abdomen that allows faeces to pass from the bowel, or urine from the urinary tract. It is the result of surgery to address conditions such as cancer, Crohn's disease or diverticulitis.

It can also be required to address bowel obstruction or injury to the digestive or urinary system.

The three main types of stoma

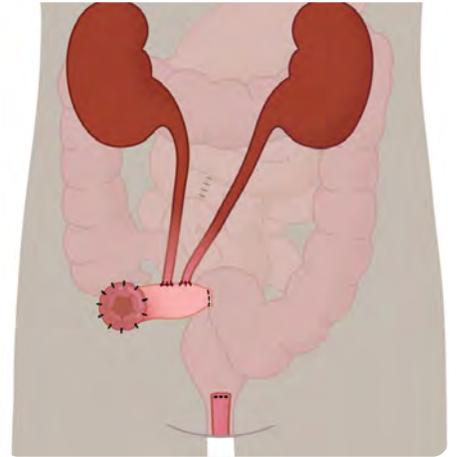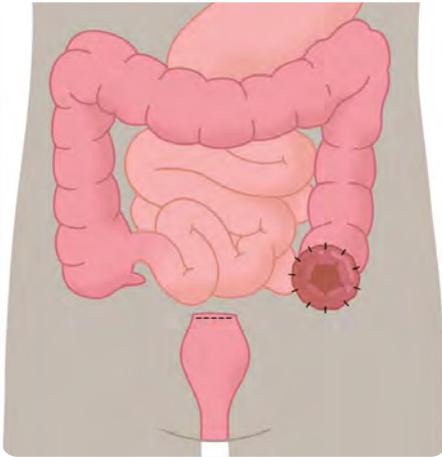

Colostomy

In a colostomy procedure, a part of your colon (the large bowel) is brought to the surface of your abdomen to form a stoma. The output from a colostomy is usually firm.

The surgical procedure

An end colostomy is usually formed when parts of the large bowel or rectum are removed. It can be a temporary or a permanent solution. If the rectum is removed, the stoma will be permanent.

A loop colostomy may be formed to protect a surgical connection in the bowel. It consists of two openings that are connected to one another. It is intended to be temporary.

Ileostomy

In an ileostomy procedure, a part of your ileum (your small bowel) is brought to the surface of your abdomen to form a stoma. The output from an ileostomy is usually more liquid than firm.

The surgical procedure

An end ileostomy is usually formed when a part of your large bowel is removed (or simply needs to rest) and the end of your small bowel is brought to the surface. It can be a temporary or a permanent solution. If the rectum is removed, the stoma will be permanent.

A loop ileostomy may be formed to protect a surgical connection in the bowel. It consists of two openings that are connected to one another. It is intended to be temporary.

Urostomy

If your bladder or urinary system is damaged or diseased and you are unable to pass urine normally, you may need a urinary diversion. This is called a urostomy, an ileal conduit or a Bricker bladder.

The surgical procedure

In a urostomy procedure, the bladder is removed and the ureters from your bladder are connected to a short length of your small bowel (ileum).

This is then diverted through an opening in your abdomen. The term 'ileal conduit' describes surgery in which a part of the small bowel (the ileum) diverts urine out of the body, by-passing the bladder.

How to care for **your stoma**

Getting a stoma is in many ways like getting a completely new body part - it does take a while to get used to something “new” being there. The days after a stoma surgery can be challenging. You have an ostomy bag attached to your abdomen and lots of new things to learn. It’s important to remember that it takes time to adjust, and that it will get easier.

Establishing good routines when it comes to your stoma is important. With good routines you decrease the risk of skin irritations and leakage which are common challenges that people living with a stoma experience.

76%

of people living with a stoma have experienced leakage within the last month

88%

of people using ostomy products have peristomal skin complications



*“Early on I didn’t have
the right fit and I would
have leakage problems”*

Mike



SenSura[®] Mio

Follows shapes and movement



Coloplast

The fit is the *difference*

SenSura® Mio

A baseplate that follows body movement



SenSura® Mio Convex

A flexible Convex with proven leakage reduction



Why SenSura Mio is *great choice for you*



The filter Improved sleep with a unique full-circle filter

A longer-lasting filter reduces ballooning by up to 61%,* to help you sleep through the night.



The adhesive of the baseplate BodyFit Technology®



The unique elastic adhesive technology, called BodyFit Technology®, provides secure body contact for protection and a flexible fit during movements.



Inspection window

Lift the bag fabric to inspect your stoma for easier application.



Discreet colour

The neutral-coloured, woven textile bag is designed to stay unnoticed under all clothing colours – even white. 94% of users are pleased with the discretion of the bag during use.¹

SenSura[®] Mio

A flat baseplate that follows body movement



Provides freedom of movement

BodyFit Technology[®] is the foundation for getting the right fit. This elastic adhesive technology is unique to SenSura Mio. BodyFit Technology provides secure body contact for protection and a flexible fit during movement




SenSura® Mio Convex

A flexible & stable Convex baseplate with proven leakage reduction



The integrated flexlines and the convex shape of the SenSura Mio Convex baseplate helps to protrude the stoma. It provides extra support while responding to the body's natural movements.

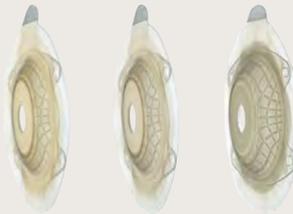

Soft

Light

Deep




A regular/flat area around the stoma

The area around the stoma is more or less even with the abdomen.




Product solutions for a regular area around the stoma

If the area around your stoma is more or less level with your abdomen, you might benefit from a flat product.

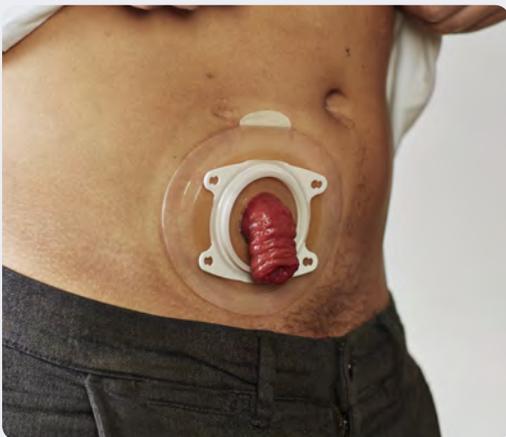

An inward area around the stoma

The area around the stoma sinks into the abdomen creating a hollow.

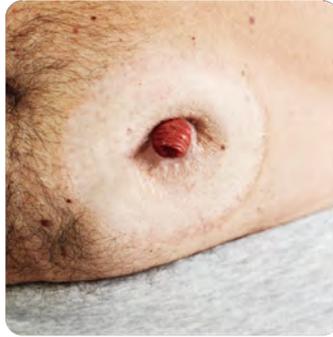

Product solutions for an inward area around the stoma

If the area around your stoma sinks into your abdomen, you might benefit from a convex product.



Find *the right fit* for you with SenSura[®] Mio

Getting used to a stoma is challenging enough without the added complexity of having to learn a different appliance system.

The new SenSura[®] Mio appliances means that patients only have to learn one, giving them more time and energy for other changes in their life.

What's more, a single range covering patients' needs from hospital to home will also save you time when it comes to training.

1-piece

SenSura Mio flat



SenSura Mio Convex



2-piece, Click

SenSura Mio flat



SenSura Mio Convex



Ensuring the *right transition* with SenSura[®] Mio

Already one of the broadest range of stoma care solutions, the addition of the new High Output and Post Op appliances makes SenSura[®] Mio a truly comprehensive range.

Healthcare professionals can now provide tailored, secure solutions for their patients at every stage of their ostomy journey.

1-piece High Output

– available with SenSura Mio convex or flat baseplate



Transparent with scale



2-piece High Output

– Click coupling to fit all SenSura Mio coupling sizes



Transparent with scale



Post-operative – sterile and non-sterile appliances featuring new packaging

Sterile



Non-sterile



The fit is the difference

between one-size-fits-all and finding the perfect solution

NEW
max-cut
sizes

SenSura® Mio is now available in new, larger max-cut sizes for patients with **large, complicated or loop stomas**, and newly operated patients whose stomas are still resolving.



All SenSura Mio baseplates are designed with BodyFit Technology – an elastic adhesive technology that provides:

- Secure body contact for protection
- Flexible fit during movement




SenSura® Mio

1-piece High Output appliance – for ileostomy patients and post-operative use

Only SenSura® Mio appliances
feature BodyFit Technology® for
security and comfort.

Now your patients can
benefit from day one.

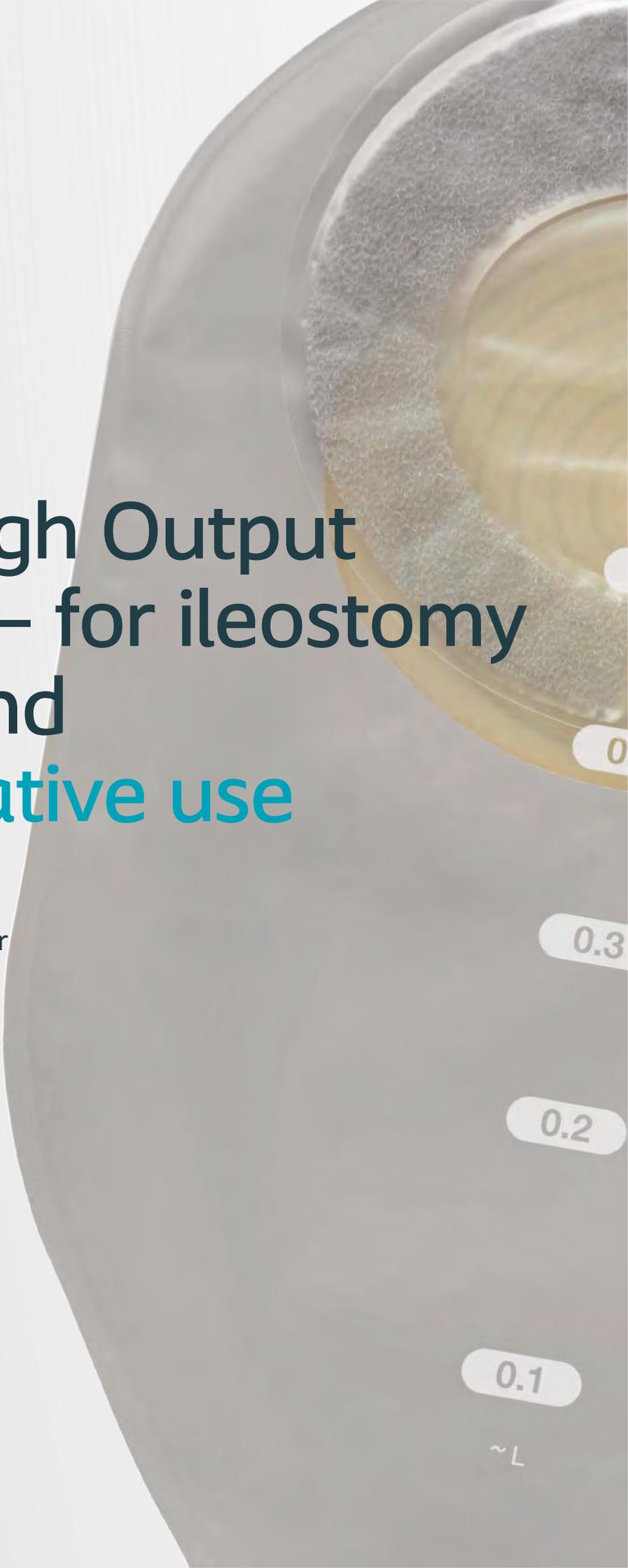



SenSura[®] Mio

2-piece High Output appliance – for ileostomy patients and post-operative use

Only SenSura[®] Mio appliances
feature BodyFit Technology[®] for
security and *comfort*.

Now your patients can benefit
from day one.

SenSura®

Dual protection - less leakage,
healthier skin




Tough on leakage. Gentle on skin.

The adhesive baseplate

SenSura's dual protection reduces the risk of leakage and improves peristomal skin complications *

The SenSura double-layer adhesive consists of a skincare layer to protect the skin from excess moisture and a security layer to protect the skin from output

- 1 Skincare layer
- 2 Security layer
- 3 Bag
- 4 Filter

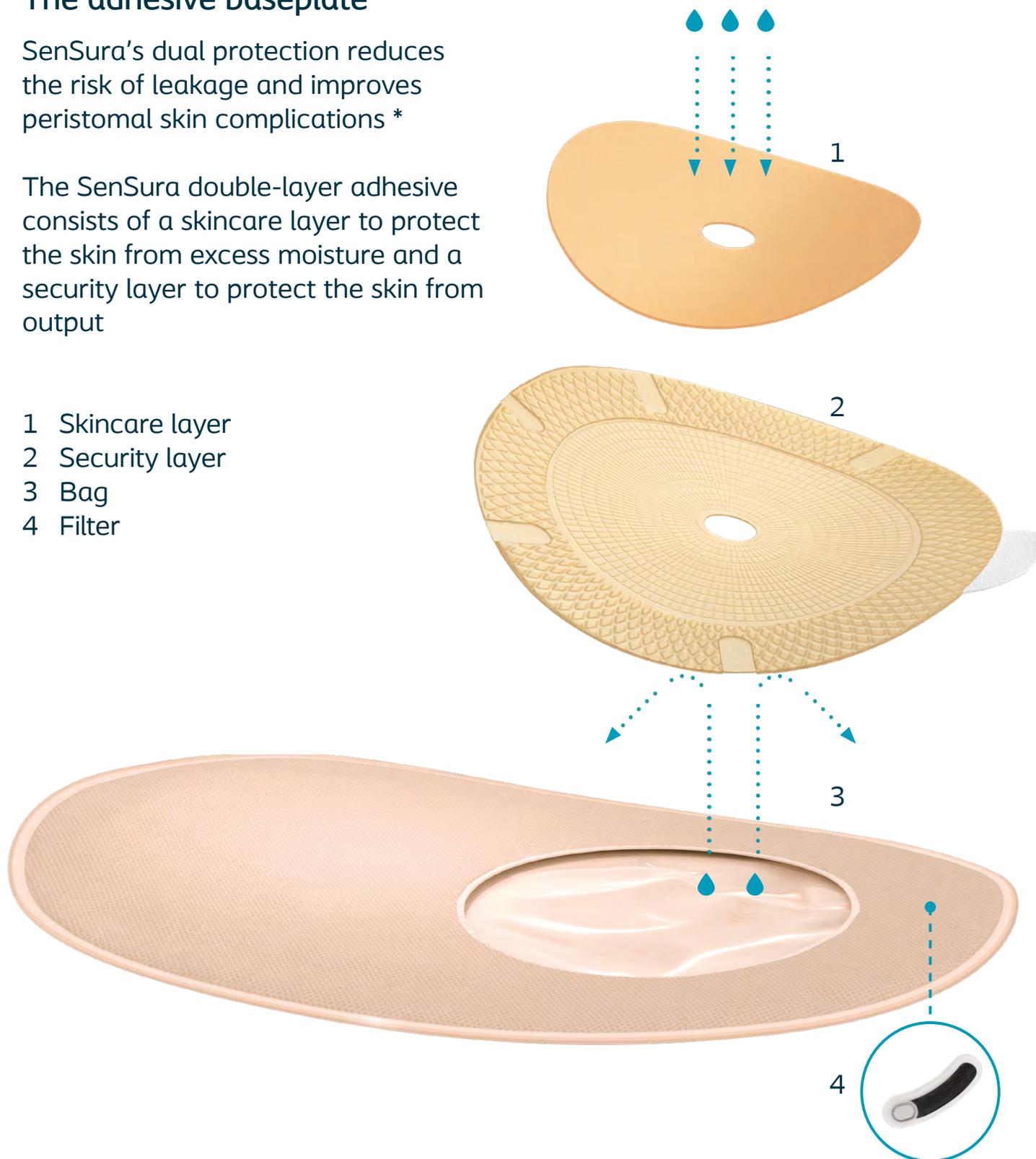

*As a combined ostomy product and study effect



The SenSura bag

SenSura is available in one-piece, two-piece, open and closed bags

All SenSura bags are made of a **non-woven material**, which makes them comfortable to wear




The SenSura filter

SenSura features an integrated filter with a pre-filter to neutralise odour and minimise the negative impact of output reaching the carbon filter

The **pre-filter** is designed to maintain airflow by preventing the carbon filter from clogging up. It also protects the carbon filter from output.

The **carbon filter** reduces the risk of ballooning and odour from the bag compared to standard of care *

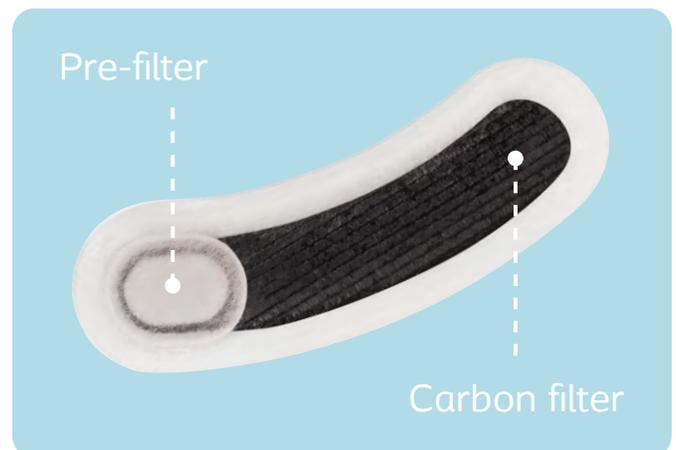

The SenSura[®] range

One piece

SenSura (flat)



SenSura Convex Light




SenSura Convex Deep



Two piece, [click](#)



Two piece, [flex](#)

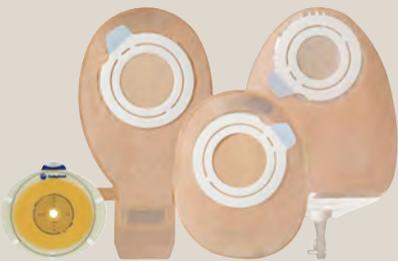

The open bag has a **hideaway outlet** for discretion, comfort, and security

The hide-away outlet can be folded away under the soft cover, which makes it discreet and comfortable to wear

The **outlet** is easy to use, clean, empty, and close

It provides **secure seal**, which keeps output secured inside the bag



SenSura[®] Post-Op bags

Post-operative Care


SenSura[®] post-op makes it possible for ostomates to feel secure right from the start, as it combines the SenSura[®] double-layer adhesive with the essentials in post-operative care.

For easy access to the stoma

Easy access is provided to the stoma through a window.

For effective drainage

SenSura[®] post-op bags allow for effective drainage through a soft outlet that connects easily to various night bags.

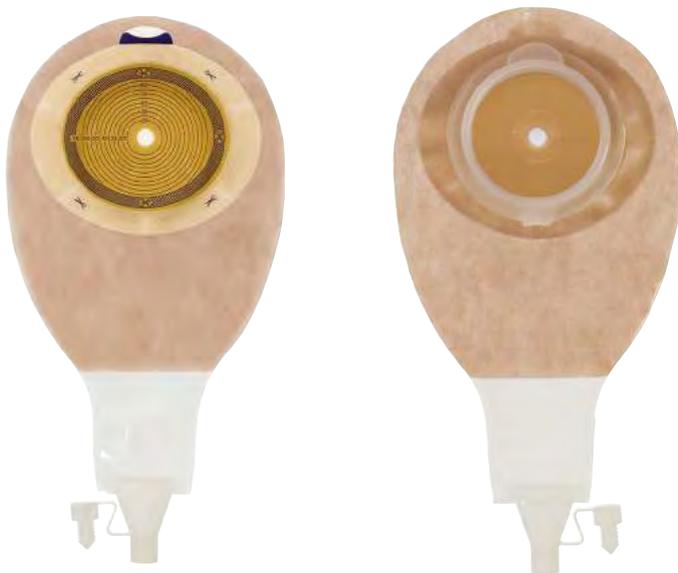

SenSura[®] post-op is available in a variety of options to meet diverse post-operative needs:

Sterile and non-sterile pouches, and even a drainable pouch without a window.

Brava®

Dual protection - less leakage, healthier skin



What's right, for you?



Brava® Supporting products

customises your
ostomy solution to
your needs



Brava® Supporting products can help you to:

Get a right fit to your body



Elastic Tapes



Protective Seal



Protective Seal Convex



Paste



Belt

Protect your skin



Skin Barrier



Barrier Cream



Protective Sheet



Powder

Support your change routine



Adhesive Remover



Skin Cleaner



Lubricating Deodorant

Brava® Protective Seal

Dual protection. Easy to handle.

Can help to improve the fit of your ostomy solution by sealing the gap between the stoma and the baseplate.

Proven to significantly reduce leakage compared to other seals



Product name	Product Code	Pack Size	Thickness
Brava® Protective Seal, 18 mm	120350	30	2,5 mm
	120450	30	4,2 mm
Brava® Protective Seal, 27 mm	120370	30	2,5 mm
	120470	30	4,2 mm
Brava® Protective Seal, 34 mm	120390	30	2,5 mm
	120490	30	4,2 mm



Designed to protect against leakage

Brava Protective Seal creates a seal between the stoma and the baseplate. Polymer formulation makes the seal robust. The seal is resistant to aggressive constituents of output so it will reduce the risk of disintegrate or get broken down by the output that it comes into contact with.

Designed to protect the skin

The ring also helps to maintain healthy skin in two distinct ways. Firstly, it absorbs moisture from the skin surface. Secondly, when removed, it leaves minimal residue behind. This makes the skin area easier to clean.

Brava® Adhesive Remover

Quick and easy removal of adhesives

Reduces the risk of trauma to the skin. It can easily remove baseplates, and adhesive – without compromising the adhesion of the next baseplate.



89%
less peel force

Product name	Item number	Unit in package	Volume
Brava® Adhesive Remover spray	120105	1	50ml
Brava® Adhesive Remover wipe	120115	30	–



Sting-free

As Brava Adhesive Remover does not contain alcohol, it does not sting upon application.

Skin friendly

Brava Adhesive Remover contains silicone, which is gentle to the skin.

Dries in seconds

There are no inconvenient delays before you can apply a new baseplate.



Brava® Elastic Tape

Appliance security for all shapes and sizes

Soft elastic tape gives extra security by keeping the appliance more stable

Skin creases, folds and moisture all have a negative effect on keeping your patients' baseplate in place. By following body shape and movement, Brava® Elastic Tape keeps the baseplate secure. It also prevents the edges of the baseplate from lifting and rolling.

Brava® Elastic Tape comes in a variety of shapes to fit various body profiles and types of adhesives. The range also includes Brava® Elastic Tape Straight for square baseplates and Brava® Elastic Tape Belt, which provides extra support for outward areas such as a hernia.



More freedom

Brava® Elastic Tape is developed with new BodyFit Technology®. This means greater security, longer wear time by preventing the baseplate edges from lifting – and more freedom of movement for your patients.

Durable and skin friendly

Brava® Elastic Tape absorbs moisture, which makes the baseplate adhesive less prone to detach from the skin.

Flexible usage

Patients can use one, two or more pieces depending on how much security and support they require.

Product name	Item number	Unit in package
Brava Elastic Tape	120700	20
Brava Elastic Tape Belt	120720	20
Brava Elastic Tape Straight	120740	20

Protects
peristomal
skin



Brava® Powder

Absorbs moisture to lessen the risk of skin irritation

Prevent skin moisture from causing problems

As well as causing a great deal of inconvenience and discomfort, skin moisture can also counteract the dry surface necessary for the adhesive to work properly. Brava® Powder absorbs moisture, keeping the skin dry and helping to stop skin irritation.

Brava® Powder can be used just around the stoma or under the adhesive.

Protects peristomal skin

Brava® Powder reduces skin irritation related to moist skin.

Helpful and friendly to skin

Brava® Powder helps to prevent red and irritated skin.



Product name	Item number	Unit in package	Size	Volume
Brava Powder	19075	1		25 g

Free of
parabens,
perfume and
alcohol



Brava® Barrier Cream

Gentle aid for dry or irritated skin

Easy-to-use and long-lasting skin protection

Brava® Barrier Cream has a moisturising effect that helps to protect against skin irritation issues from when skin comes into contact with output, or when cleaning or removing the appliance.

Brava® Barrier Cream creates a long-lasting shield for the skin that ensures the baseplate adheres to the body. It also creates a long-lasting barrier that shields the skin from further damage from stoma output or adhesives.

No parabens, perfume or alcohol

Brava® Barrier Cream is free of these potential irritants, making it gentle as well as effective.

Does not compromise tapes or adhesives

Brava® Barrier Cream is designed to fit into your patients' appliance change routine. It allows tapes and adhesives to adhere so it does not need to be removed before applying the bag.

Effective moisturising

Because Brava® Barrier Cream moisturises, it helps to manage sore, dry or irritated skin and prevent further damage.

Product name	Item number	Unit in package	Volume
Brava Barrier Cream	120005	1	60 ml

Provides a
feeling of
security



Brava® Belt

Get a better fitting appliance – and more peace of mind

The secure way to a better fit

The Brava® Belt discreetly provides extra security by helping to keep the baseplate in place.

Made of soft and comfortable material, the Brava® Belt comes in both standard and XL sizes that are adjustable to fit almost any body shape.



Simple to wear

The Brava® Belt fits with all SenSura® and Assura® baseplates with belt ears.

There is a special belt for SenSura® Mio baseplates.

Hygienic and reusable

The Brava® Belt is designed to be used multiple times and can be washed at 60 °C or higher.

Help for all body shapes

The Brava® Belt can be particularly useful for those looking for the security of a more stable appliance.

Product name	Item number	Unit in package	Size
Brava Belt	04215	1	Standard
Brava Belt	04220	1	XL
Mio Belt	00423	1	Standard
Mio Belt	00424	1	XL

Sting-free
paste



Brava® Paste

Kind on skin, tough on leakage

Durable protection against leakage

Even if your patients have difficult folds and creases in their skin, Brava® Paste helps them to stay on top of leakage issues. Brava® Paste fills out difficult skin contours, maintaining a tight fit between the stoma and the appliance.

Brava® Paste is alcohol free so there is no itching or burning sensation when applying.

No sting

Brava® Paste does not contain alcohol so there is no stinging sensation when it is applied.

Shields skin

Brava® Paste helps to prevent skin issues by protecting against output. It also has a skin-friendly pH level.

Resistant to breaking down or dissolving

Because Brava® Paste is so durable, it keeps working to help skin stay healthy.

Product name	Item number	Unit in package	Size	Volume
Brava Paste	120500	1		60 g

Easy
to use



Brava® Strip Paste

The filler that improves peristomal skin

A tight seal despite scars and folds

The human body rarely provides stoma patients with the ideal surface for an appliance. Natural contours of the skin can lead to adhesion and leakage issues. Brava® Strip Paste can fill out even creases and deep folds, absorbing moisture and creating an even surface for the baseplate. This gives a better seal between the baseplate and skin, which helps reduce the risk of leakage.

Flexible, durable and strong

Stoma users can be worried that their skin surface is too uneven or moist to create a strong seal. Brava® Strip Paste helps improve the peristomal skin. All that is necessary to do is to tear off the required length of strip and mould it into the required shape.

Product name	Item number	Unit in package
Brava Strip Paste	026555	10

Skin
friendly



Brava® Protective Sheet

An extra layer of skin protection

Gentle peristomal skin protection

Patients with moist skin are often vulnerable to skin maceration and its attendant discomforts. The skin-friendly Brava® Protective Sheet is used under the baseplate, providing extra absorption in addition to the existing ostomy solution.

The Brava® Protective Sheet comes in a variety of sizes to fit different body shapes and needs.

Creates a solid foundation

The Brava® Protective Sheet helps to avoid skin issues by keeping skin dry and providing a ready surface for applying a new appliance.

Product name	Item number	Unit in package	Size
Brava Protective Sheet	032105	10	10x10 cm
Brava Protective Sheet	032155	5	15x15 cm
Brava Protective Sheet	032205	5	20x20 cm

Dries in
seconds



Brava® Skin Barrier

An invisible friend for the skin

Shields against output and adhesives

Leakage and adhesives can cause patients a variety of skin problems. Brava® Skin Barrier leaves a thin breathable film on the skin that protects against potential damage but does not affect the adhesion of the next appliance.

The Brava® Skin Barrier is a silicone-based product so it does not sting when applied. It comes as a spray and a practical travel-friendly wipe.

Prevents residue build-up

When your patients use Brava® Skin Barrier, residue either comes off when removing the adhesive or by gentle cleaning. This means it does not build up on the skin over time.

Dries in seconds

With Brava® Skin Barrier, your patients will be ready to apply a new adhesive almost immediately.

Product name	Item number	Unit in package	Volume
Brava Skin Barrier spray	120205	1	50 ml
Brava Skin Barrier wipe	120215	30	-

Odour
reduction



Brava® Lubricating Deodorant

Multi-lubricating effect

Deodorant and lubrication in one

Odour from the output or output gathering around the stoma is a challenge for many patients. Specially designed to be used inside the bag, Brava® Lubricating Deodorant helps against odour when changing or emptying an appliance, while the multi-lubricating effect makes it easier for output to go to the bottom of the bag.

Brava® Lubricating Deodorant comes in a 240 ml bottle and an easy single-use sachet, which is ideal for patients who are away from home.



Smooth removal and emptying

As well as making the bag easier to empty, Brava® Lubricating Deodorant's multi-lubricating effect also helps to keep the content at the bottom of the bag rather than around the stoma.

Bag seal and adhesive still strong

Brava® Lubricating Deodorant is optimised for use with ostomy appliances so it does not degrade bag seals or affect the function of the adhesive and bag.

Skin friendly

Although Brava® Lubricating Deodorant is designed to be applied inside the ostomy pouch, it is a skin-friendly product so there are no issues with it coming into contact with skin.

Product name	Item number	Unit in package	Size	Volume
Brava Lubricating Deodorant	120610	1		240 ml
Brava Lubricating Deodorant	120600	20	Small	7.5 ml



Fistula and Wound Management System

Environmental and disposal information

Composition of product

- Ethylene vinylacetate - EVA
- Polyethylene - PE
- Styrene-ethylene-butadiene-styrene - SEBS
- Polyethylene foam - PE
- Polyvinylidene chloride - PVdC
- Polyester - PET
- Polyolefin - PO
- Adhesive
- Potato starch
- Pigment
- Activated carbon
- Methylparaben
- Ethylparaben

Accessory - PUMP

- Polyoxymethylene - POM
- Polypropylene - PP
- Polyisoprene - PI

Accessory - Cover sheet, tracing guide

- Ethylene vinylacetate - EVA
- Polythylene - PE

Accessory - CLAMP

- Polypropylene - PP

Composition of packaging

The retail and transport boxes are made of cardboard.

- Carrugated cardboard
- Polyester - PET
- Polythylene - PE

Product Code	Product Name	Pack Size
140100	Bed drainage bag	10
140150	Drain part	12
140200	Flexible lid, mini, with filter	10
140300	Flexible lid, midi, with filter	10
140400	Flexible lid, maxi, with filter	10
140550	Fistula and Wound Mng System, mini (with filter)	6
140650	Fistula and Wound Mng System, midi (with filter)	3
140750	Fistula and Wound Mng System, maxi (with filter)	3



"I feel confident. I didn't expect such a small product to make such a big difference"

Louise about Brava® Protective Seal

Making life with a stoma easier_

The information provided is to be used only as a guide and should not be relied upon as medical advice..

011 700 5000
dtcsa@coloplast.com
www.coloplast.co.za



Coloplast and the Coloplast logo are trademarks of Coloplast A/S.
© 2026-03 Coloplast A/S. All rights reserved.