

It's All About...

T R U S T

Conveen

Self-Sealing Urisheath

A one-piece urine collecting system

- Easy and secure application
- Can be applied while wearing gloves
- Dependable adhesive protects the skin
- Bulb port allows unobstructed flow





Conveen




Conveen Self-Sealing Urisheath

With its absolute dependability and quick, simple and hygienic application, the **Conveen** Self-Sealing Urisheath offers advantages to users as well as to caregivers. Like all other **Conveen** products, it offers optimal comfort and skin-friendliness.

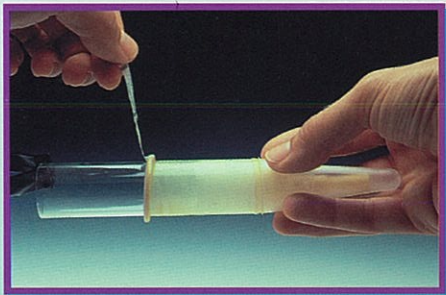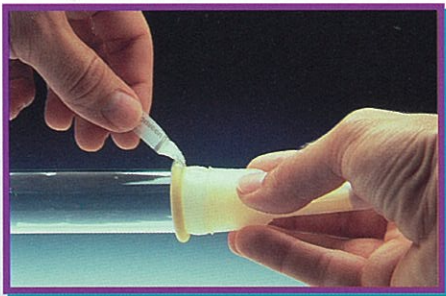


Easy and secure application

A convenient applicator makes it easy to apply the urisheath correctly. It can be done while wearing gloves to avoid direct contact with the penis. Pre-fitted to the applicator, the urisheath is held over the penis. By gently pulling the tab end of the unrolling strip, the talcum-coated urisheath unrolls smoothly. A light pressure on the area around the clearly visible self-contained adhesive secures adherence to the penis. The application procedure is so easy that even those with limited dexterity can manage it without difficulty. Those who wish to do so can apply the urisheath without using the applicator and unrolling strip.

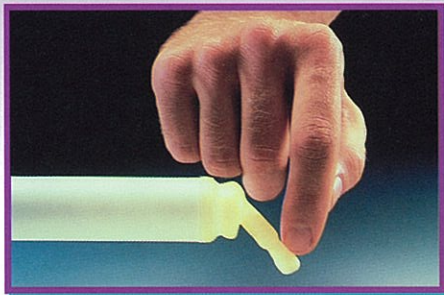

Unobstructed flow

The self-sealing urisheath features a bulb port that ensures the flow of urine, even when it is bent and twisted. It can be bent 90° in any direction, and the anti-kink bulb port will continue to perform with complete dependability.

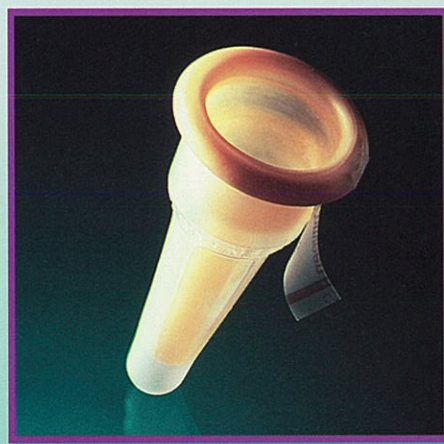

Dependable and skin-friendly adhesive

The self-contained adhesive provides a leak-proof seal for 24 hours. As it is just as flexible as the urisheath itself, the adhesive feels comfortable and will not tighten around the penis. The adhesive is both secure and skin-friendly.

Optimum reliability

The **Conveen** Self-Sealing Urisheath is made of skin-friendly and high quality materials. Four sizes are available; thus optimum reliability is ensured by a proper fit.



Conveen Collecting System

The **Conveen** Self-Sealing Urisheath is part of a urine collecting system that includes **Conveen** Urisheath/Urilliner, **Conveen** Security+ Urine Bag and **Conveen** 1500 ml Urine Bag.



PRODUCT RANGE

Product code	Ø/mm	Circumf./mm	Units per box
052000	25	75	30
052050	30	90	30
052100	35	105	30
052150	40	125	30



Coloplast

Coloplast S.A.

P. O. Box 505, Wendywood, 2144

Tel: (011) 802 2943 Fax: (011) 802 2924

Tollfree : 0800 003 425

Co. Reg. No. 94/08553/10

It's All About...

T R U S T

Conveen

Security+ Urine Bag

A discreet, trustworthy and comfortable leg bag

- Corrugated inlet tube gives free passage to urine in all positions and adds to discretion
- Centre welding gives filled bag a low profile
- Secure and easy-to-operate outlet valve

Conveen




Conveen Security+ Urine Bag

In choosing a urine bag, no criteria are more important than the security and discretion it offers. The **Conveen Security+ Urine Bag** offers this and more.



Always free passage


No matter how the corrugated inlet tube is bent or twisted, it remains kink-free, making backflow impossible. Moreover, the exceptional flexibility of the tubing enables it to follow the natural movements of the leg, making the tubing just as discreet as the bag itself. If it is too long, the tubing can be shortened with a pair of scissors. The smooth parts of the inner wall of the tubing prevent the upward migration of bacteria, thereby ensuring hygienic protection.

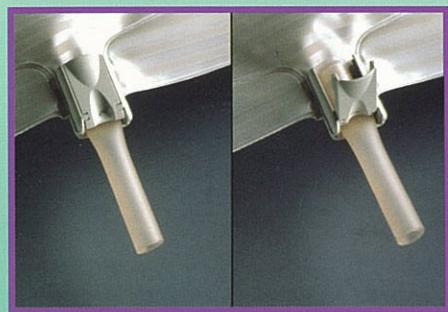

A safe outlet valve that is easy to operate

A low profile

The bag is centre-welded, creating a partial inner barrier that allows the urine to be evenly distributed into two pockets. This has a twofold function. It flattens the bag's profile, making it less visible, and it prevents the revealing sound that can be caused by urine splashing in a partially filled bag. It's simply more discreet.

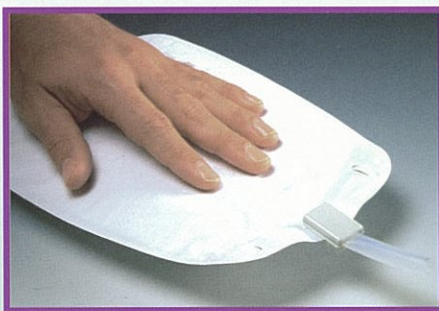

A safe outlet

The outlet valve provides a tight and secure seal. To prevent mistakes, the valve is designed so that it is easy to see if it is properly closed. Adding to this security, an audible click indicates complete closure. The valve is easy to operate, even for those whose dexterity is limited.

Other quality features for security and comfort

- Double weldings for double security
- The non-return valve prevents backflow, making the bag more hygienic
- Secure connection to the urisheath
- Soft, non-woven backing allows the skin to breathe and it feels comfortable
- The bag, made without PVC, is soft and light
- Adjustable leg bag straps available separately to hold the bag securely and comfortably in place



Conveen Collecting System

The **Conveen Security+ Urine Bag** is part of a urine collecting system that includes the **Conveen Urisheath/Urilinear**, the **Conveen Self-Sealing Urisheath** and the **Conveen 1500 ml Urine Bag**.

PRODUCT RANGE

	Product code	Tube length/ cm	Volume/ ml
	051650	50	350
	051610	50	500
	051670	50	750
	5050	Leg bag straps	

 **Coloplast**

Coloplast S.A.

P. O. Box 505, Wendywood, 2144

Tel: (011) 802 2943 Fax: (011) 802 2924

Tollfree : 0800 003 425

Co. Reg. No. 94/08553/10

It's All About...

TRUST

Conveen

Urisheath/Uriliner

A two-piece urine collecting system

- Uriliner expands and contracts along with penis
- Hydrocolloid adhesive protects the skin
- Anti-kink outlet ensures unobstructed flow





Conveen




Conveen Urisheath/Uriliner

The most reliable product of its kind available today. It incorporates an exclusive combination of functional advantages, convenience, comfort and economy. The **Conveen** Urisheath/Uriliner can be used with complete confidence.

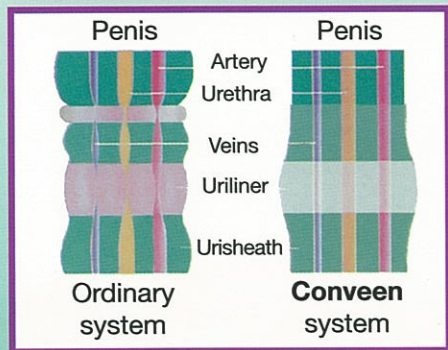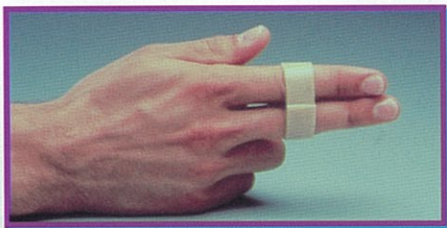

Resilient uriliner

The uriliner - a double-sided adhesive strip - keeps a firm yet nonconstricting grip on both penis and urisheath. It means that the urisheath stays absolutely leak-proof for 24 hours, even though the uriliner may expand and contract many times along with the penis. Due to the "memory" feature of the material, the uriliner will always return to its original size and tension.

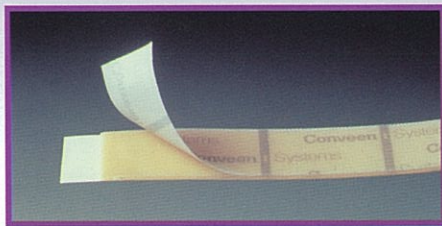


Skin-friendly adhesive

The hydrocolloid adhesive absorbs moisture and offers reliable protection against skin irritation and maceration. Slight variations in the placement of the uriliner from one application to the next will give the skin a rest between applications. And since the resilience of the uriliner allows an uninterrupted flow of blood to the penis, there is no danger of pressure sores.


Release tabs

The protective cover paper on the uriliner extends past the adhesive, making it easy to grasp and simple to remove.

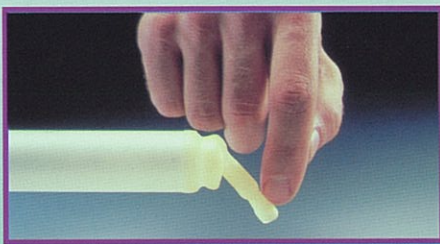

Unobstructed flow

The anti-kink outlet ensures the flow of urine, even if it is bent and twisted. It can be bent 90° in any direction without blocking the flow of urine.

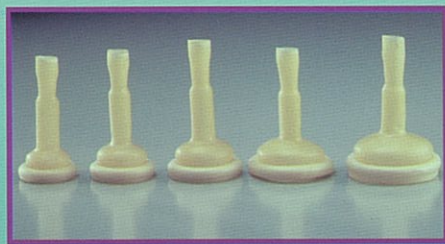

Optimum reliability

The **Conveen** Urisheath/Uriliner is made of high quality, skin-friendly materials. Four sizes are available ensuring optimum reliability by a proper fit.



Conveen Collecting System

The **Conveen** Urisheath/Uriliner is part of a urine collecting system that includes the **Conveen** Self-Sealing Urisheath, the **Conveen** Security+ Urine Bags and the **Conveen** 1500 ml Urine Bag.



PRODUCT RANGE

Product code	Ø/mm	Circumf./mm	Units per box
051250	25	75	30
051300	30	90	30
051350	35	105	30
051400	40	125	30



Coloplast

Coloplast S.A.

P. O. Box 505, Wendywood, 2144

Tel: (011) 802 2943 Fax: (011) 802 2924

Tollfree : 0800 003 425

Co. Reg. No. 94/08553/10